NEW

DAN = LOC

FROM DAN-LOC



FFP2 **PROTECTIVE** MASK

FEATURES & BENEFITS

- Two-strap design with adjustable nose clip provides a secure seal
- Comfortable nose foam cushion

PRODUCT DESCRIPTION

FFP2 PROTECTIVE MASKS are used to protect the wearer from airborne particles and from liquid contaminating the face. These single-use masks provide nose and mouth coverage.

PACKAGING

ltem	иом	Minimum Order Quantity (Orders should be in multiples of min)	Carton Size (cm) [inches]	Gross Weight (kg) [lbs]
FFP2 Protective Mask	Each	250	(38 / 32.5 / 32) [15 / 12.8 / 12.6]	(3.5) [1.58]

WE PERFORM SO YOU CAN PERFORM.



FFP2 PROTECTIVE MASK TECHNICAL DATA





DANLOCGROUP.COM MedicalSupplies@danlocgroup.com

725 Drennan St. Houston, TX 77003 +1 800-231-6355 +1 713-356-3500

CTANDARD	
STANDARD	EN 149:2001
FILTER EFFICIENCY	≥ 94 %
NARRATIVE	EN149:2001 are designed to protect against solids, water based aerosols and oil based aerosols. Filter efficiency of the filtering medium: 94% Total Inward Leak: < 8% Allocated Protection Factor (FPA): 10
STORAGE	Store in a cool, dry place away from frost and light in their original packaging (-30°C/+70°C -Relative Humidity <80%).
WARNING	Use the mask in well ventilated areas, where there is no lack of oxygen (> 19.5%) and where there is no explosive atmosphere. The mask must be positioned correctly in order to provide the expected level of protection. Do not use if facial hair (beard, sideburns) prevents the mask from being fully sealed, and leakage is detected. Leave the area immediately if you experience breathing problems and/or dizziness or faintness. Do not make any modifications to the mask, and discard it if there is any damage to it. These dust masks DO NOT PROTECT the user against gases or vapors and solvents from spray painting. Failure to observe these instructions for use may have adverse effects on the health of the user and expose them to serious or even life threatening risks.

MedicalSupplies@danlocgroup.com | www.danlocgroup.com